



www.techppe.co.uk

Haika HKC01 Reusable Surgical Gown









EN13795-1

EU Compliance

- Class 1 Medical Device
- EN13795-1 Standard Performance

Ordering Information

• Product Code: HKC00001DZ

• Pack Quantity: 1 single per bag

• Case Quantity: 10 singles per carton



www.techppe.co.uk

Haika HKC01 Reusable Surgical Gown



Storage and Maintenance

Wash Instructions

- Non-ionic detergent should be used
- Do not use bleach
- Do not iron
- Do not dry clean
- Wash at a maximum temperature of 73°C
- Tumble Dry exhaust temperature should not exceed 60°C. Controlled cooling, after wash, to 40°C, will minimise creasing
- Remove load immediately once goods are dry
- Maximum 100 washes

Storage

 It is recommended that the garments are stored in their original packaging clear of UV sources. Following use and cleaning, garments should be stored in a cool, dry and clean environment.

Components

 Body: Bakewell (Toray), 99.3% Polyester, 0.7% Carbon Graft

• Cuff: Polyester

• Thumb Loop: Braided Elastic

• Tie: Polyester Herringbone Weave

• Binding: Polyester Blend

Sizing

• Item: One size

Center Back Length (cm): 120

• Width (cm): 160

• Under Arm Length including cuff (cm): 64.5



A design collaboration between ICL Tech Ltd and Globus Group. Manufactured by Globus in the UK // Distributed by ICL Tech Ltd.



Supplied by ICL Tech Ltd, 1 Wellington Road, Bishopbriggs, Glasgow, G64 2SA